Armon Pura

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Liability:

To the maximum extent permitted by applicable law, in no event will Armon Products or its suppliers or resellers be liable for any indirect, special, incidental or consequential damages arising from the use of or inability to use the product, including, without limitation, damages for loss of goodwill, work and productivity, computer failure or malfunction, or any and all other commercial damages or losses, even if advised of the possibility thereof, and regardless of the legal or equitable theory (contract, tort or otherwise) upon which the claim is based. In any case, Armon Products's entire liability under any provision of this agreement shall not exceed in the aggregate the sum of the fees paid for this product and fees for support of the product received by Armon Products under a separate support agreement (if any), with the exception of death or personal injury caused by the negligence of Armon Products to the extent applicable law prohibits the limitation of damages in such cases.

Armon Products cannot be held liable for any consequence resulting from incorrect information provided by its personnel, or errors incorporated in this manual and / or other accompanying documentation (including commercial documentation). The opposing party (product's user or its representative) shall disclaim Armon Products from all claims arising from third parties, whatever nature or whatever relationship to the opposing party.

Any severe incident in relation to this product should be mentioned to the manufacturer and the competent national authority

Warranty:

Warranty claims shall be voided in case:

- The conditions in the operating manual are ignored
- The appliance is used outside the described uses
- Any part of the appliance is modified or opened
- The appliance is improperly set up or incorrectly electrically connected

Cleaning:

When cleaning the outer part of the Armon Pura and the brace, make sure to use a soft, clean cloth dampened with lukewarm water and a non-abrasive liquid household cleaner.

**CAUTIONS:**

- When cleaning the outer part of the Armon Pura, do not use any chemicals, detergents, acid solutions, abrasive or aggressive solutions or inflammable substances.
- Use of any of the above mentioned substances or improper use of the accessories will cause irreparable damage to the equipment and the guarantee will be no longer valid. Do not spray or pour liquid on the external parts of the equipment, or into the holes on the casing.
- Do not immerse the unit into water.
- The unit must under no circumstances be opened or dismantled in order to clean or check inner parts – the Armon Edero does not require cleaning of inner parts and in all cases, only specialized technicians of Armon Products B.V. should carry out such operations.

Durability and maintenance:

- **Expected technical lifetime:**

The Armon Pura is designed to guarantee a technical lifetime of at least 6 years. Depending on the intensity of use and the environmental circumstances (dirt, dust, sea air, humidity etc.) the lifespan can be shortened or extended.

- **Maintenance:**

The Armon Pura is in general a maintenance free product within the expected technical lifetime. However, wear and tear of the moving internal mechanics depends on the intensity of use and the environmental circumstances (dirt, dust, sea air, humidity etc.). Careless use or disregarding normal clean handling principles of similar products can lead to early repairs.

Inside parts can be replaced preventively in order to keep the product functioning with a high level of security. Contact your distributor for possible preventive maintenance contract terms and costs.

End of life:

The Armon Pura contains materials that can be recycled and/or are noxious to the environment. Specialized companies can dismantle the unit and sort out these materials. When you dispose of the unit, find out about local regulations concerning waste management.

Technical data:

<i>Weight Armon Pura:</i>	6100 grams (including basic brace)
<i>Weight compensation range:</i>	1000 till 4180 grams +/- 10%
<i>Housing size WxDxH (in upwards position)</i>	17x10x85 cm
<i>Ingress Protection / International Protection rating:</i>	Pura: IP 15 Brace: IP 45
<i>Housing material:</i>	Die cast aluminum and ABS plastic
<i>Unique identification of the object:</i>	SN: PU.... (product and serial number)
<i>Description of object:</i>	Fully mechanical arm support.
<i>The object complies with the following directive:</i>	93/42EEC Medical Device Directive
<i>The object complies with the following Standards:</i>	EN ISO 22523:2006 External Orthoses
<i>The object of the declaration described above meets the demands for CE labeling:</i>	Medical Device Class I

Armon Products

Armon Products BV develops and manufactures high end dynamic arm supports and accessories according to the highest standards of quality making use of the latest technology and fully adhering to current directives and norms.

Particular attention has been paid to the design, easy operation, function and safety of the equipment and the final result is this modern, compact unit.

Armon stands for Innovation, Design and Durability.

Our goals are to increase upper limb mobility, make people less dependant and improve quality of life.

Armon Products B.V.
De Nestel 7, 5107RL
The Netherlands

Tel. (+31) (0)10 4714187
www.armonproducts.com
info@armonproducts.com